Instructions for use

**Cutimed® Alginate****(ENG)****Product Description**

Cutimed® Alginate is a sterile, non-medicated alginate dressing, consisting of Calcium- and Sodium-alginate. Cutimed® Alginate Dressing is made of semipermeable nonwoven fabric coated on both sides with natural fibers derived from seaweed. A hidden integrity layer in the dressing gives it high wet stability. It is designed to be highly, fast absorbent. Cutimed® Alginate absorbs wound exudate and forms a gel-like covering over the wound, maintaining a moist wound healing environment which supports the healing process and helps to stop bleeding. Gel-like consistency of Cutimed® Alginate enables an easy and painless removal from the wound and the high wet strength of Cutimed® Alginate ensures the dressing remains integral while removing.

**Intended Use and Indications**

Cutimed® Alginate is a highly absorbent wound dressing. It is indicated for moderately to highly exuding wounds, such as:

- Surface trauma
- Moderately to heavily draining wounds, cavity wounds
- Pressure sores
- Diabetic ulcer
- Venous/arterial leg ulcer
- Donor sites
- Post-surgical wounds

Cutimed® Alginate dressings are suitable for use under compression bandages.

**Contraindications**

Cutimed® Alginate dressing is not indicated for:

- Surgical implantation
- Dry wounds
- To control heavy bleeding
- To control wound infection

As allergies can never be excluded, Cutimed® Alginate should not be applied with a known or suspected allergy to one of the ingredients. In case of doubt please consult a physician.

**Wound preparation**

Debride when necessary and irrigate the wound in accordance with local clinical protocol.

Remove excess moisture from surrounding skin.

**Dressing Application**

Select an appropriate dressing according to wound size.

- (1) Cut (using sterile scissors) or fold the dressing to fit the wound.
- (2) Apply to wound bed directly. Loosely fill deep wounds, ensuring the dressing does not overlap the wound margins.
- (3) (4) Cover and secure with a suitable secondary dressing, for example Fixomull® transparent, Hypafix® transparent or Cutimed® Hydro L.

**Dressing Change and Removal**

Cutimed® Alginate can remain in situ up to 7 days, depending on patient condition, the level of exudate and the saturation of the secondary dressing. If required, change the dressing more frequently (every 1 to 3 days), depending on the wound situation. The continuous usage should not exceed 30 days in total.

Carefully remove the secondary dressing, and gently remove Cutimed® Alginate from the wound bed and discard.

If the wound appears dry, saturate the dressing with sterile saline solution prior to removal.

If required, irrigate the wound site with a suitable wound cleanser prior to application of new dressing.

**Infected wounds**

1. If the wound is infected, appropriate medical treatment should be initiated.
2. Application of the Cutimed® Alginate dressing may be continued at the discretion of the attending medical practitioner/clinician.

**Storage (and Disposal)**

Cutimed® Alginate should be stored in a cool, ventilated and dry place avoiding light.

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