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Cutimed® Sorbact® Hydroactive B

D

Produktbeschreibung

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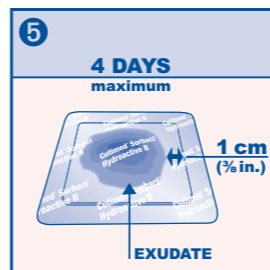
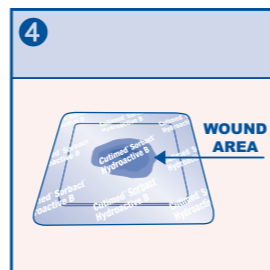
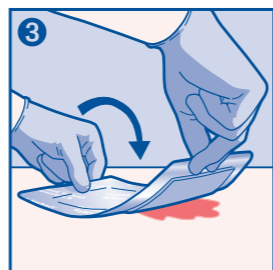
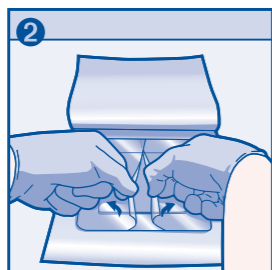
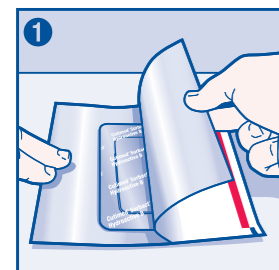
ENG

Product description

Cutimed® Sorbact® Hydroactive B is a bacteria-binding, absorbent gel dressing with an adhesive border. Cutimed® Sorbact® Hydroactive B combines a highly absorbent hydropolymer matrix with bacteria-binding Sorbact mesh (Sorbact acetate fabric coated with dialkylcarbomoylchloride – DACC) and is covered by a semipermeable polyurethane film. The fixation border is made of a latex free adhesive hydrogel.

Indications

Cutimed® Sorbact® Hydroactive B is indicated for use in the management of low to moderate exuding partial to full thickness wounds (including clean, colonized, contaminated, or infected wounds): chronic wounds (venous and arterial ulcers, diabetic ulcers and pressure ulcers), postoperative dehisced wounds and traumatic wounds.

Contraindications

Do not use Cutimed® Sorbact® Hydroactive B on dry wounds or cavity wounds. Do not use on patients with known sensitivity to the dressing components.

Warnings and Precautions

Cutimed® Sorbact® Hydroactive B is packaged for single use. Do not re-use or re-sterilise as there is a risk of cross contamination. Gel may expand. (Dark) gel discharge is not critical. Do not use if pouch is opened or damaged. Discard any open dressings. Frequency of dressing change of Cutimed® Sorbact® Hydroactive B is dependent upon the wound condition and exudate level. Cutimed® Sorbact® Hydroactive B can remain in place for up to 4 days.

When in contact with wound exudates, the highly absorbent hydropolymer matrix changes from yellow to brown, indicating the dressing has become saturated. When exudate absorption is observed at 1cm (3/8") from matrix edge, maximum absorption has been reached and dressing should be changed.

Cutimed® Sorbact® Hydroactive B is suitable for use in combination with low pressure compression therapy for contaminated, colonised or infected exuding wounds.

Cutimed® Sorbact® Hydroactive B should not be cut to shape wound. Avoid overlapping two or more Cutimed® Sorbact® Hydroactive B dressings.

Do not use Cutimed® Sorbact® Hydroactive B in combination with ointments or creams.

Directions for use

1. Clean the wound as usual.
2. Select an appropriate dressing according to wound size. For best results, make sure Cutimed® Sorbact® Hydroactive B overlaps the wound margins by at least 2cm (3/4").
3. Remove the protective film and apply the green adhesive side of the dressing to the wound.
4. Depending on the wound location it may be appropriate to apply additional dressing fixation.
5. Cutimed® Sorbact® Hydroactive B should be changed as often as dictated by the wound condition. As with all absorbent dressings, monitoring is required to ensure the dressing does not dry out and adhere to the wound. The dressing should be changed when it becomes cloudy or opaque from fluid absorption. At the latest after 4 days or if the exudate has reached 1cm (3/8") from the edge.

In wounds that show signs of clinical infection, a more frequent wound inspection is advised. In these cases, an appropriate systemic treatment should be considered.

Storage

Cutimed® Sorbact® Hydroactive B should be stored in dry conditions below 25°C/77°F. Store the dressing flat.

Presentation

Article No.	Size	No. / Box
79933-00	7 cm x 8,5 cm / 2¾ in. x 2¾ in.	10
79933-02	14 cm x 14 cm / 5½ in. x 5½ in.	10
79933-03	19 cm x 19 cm / 7½ in. x 7½ in.	10
79933-04	14 cm x 24 cm / 5 ½ in. x 9 ½ in.	10

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STERILE R **0124**
Sterilized using irradiation

Do not re-use Keep dry Keep away from sunlight Do not heat above 25°C/77°F Caution!

Made and sterilized in UK

BSN medical GmbH
Quickbornstrasse 24
20253 Hamburg · Germany

BSN medical

JBR06503.02

	Artwork ID JBR06503.02	Review Loop 3	Printing Colors  Pantone 199 C  Pantone 287 C
Product Title Cutimed® Sorbact® Hydroactive B		Date of Review Loop 22.04.2015	
Contact artwork.coordination@bsnmedical.com		Replaces Artwork ID JBR06503.01	
Legal Manufacturer BSN medical GmbH Quickbornstraße 24 20253 Hamburg		Technical Drawing for print ID -	
Ref Number(s) 79933-00, 79933-02, 79933-03, 79933-04		Comment on Diecut Die cut for control only-not for production.Please use current die cut.Proof binding only for CMYK colors.	