Instructions for Use

Cutimed® Debriclean

Product description

The wound side of Cutimed® Debriclean consists of white mono-filament microfiber loops which effectively clean the wound and protect intact tissue. The blue, abrasive microfiber loops feature effective wound cleansing and remove even mild necrosis. The mono-filament microfiber loops:

- safely and effectively remove biofilm
- bind and retain bacteria
- absorb wound exudate, slough and dry skin flakes
- are easy and safe to use with visible results after just a few minutes

Indications

Cutimed® Debriclean aids in the debridement of superficial wounds and surrounding skin in, for example:

- diabetic ulcers
- venous and arterial ulcers
- pressure sores
- postoperative wounds and wounds healing by secondary intention
- superficial burns
- the absorption of exudate, slough and dry skin flakes

Contraindications

There are no known contraindications.

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Directions for use

- Where necessary, remove the wound dressing
- Prepare the following:
 - single-use protective cover
 - single-use gloves, more than one pair where necessary
 - hand-disinfectant
 - wound-cleansing solution
 - Cutimed® Debriclean
- Performing debridement:
 - Moisten** Cutimed® Debriclean in its packaging with wound-cleansing solution (according to the national guidelines). All current wound-cleansing solutions are suitable (including antiseptics)*. Ensure the pad is **fully saturated** (recommended amount: approx. 30 ml.) **Ⓐ**
 - If the pad is insufficiently moistened, the patient may experience pain.
 - Use the strap on the back to hold Cutimed® Debriclean. **Ⓑ**
 - Use the wound side of the moistened Cutimed® Debriclean to clean the wound, using circular movements while applying gentle pressure. **Ⓒ**
 - Use the abrasive, blue area on the wound side of Cutimed® Debriclean to help remove mild necrosis.
 - Change the cleaning area as soon as the mono-filament microfiber loops have collected sufficient slough, or use a new Cutimed® Debriclean.
 - Dispose of used Cutimed® Debriclean via regular household or clinical waste.
 - After debridement proceed with the standard dressing change.
 - Treatment of the wound with Cutimed® Gel before the dressing change is helpful with the debridement of necrotic wounds.
 - Cutimed® Debriclean shall be applied by healthcare professionals only.

*current wound-cleansing solutions based on PHMB, sodium hypochlorite or hypochlorous acids, octenidine, 0.9% medical saline solution, water and Ringer’s solution

Storage and Disposal

Store in dry surroundings, keep away from sunlight and protect against high air humidity. Keep out of the reach of children. Disposal should be made according to local regulations.

Notice

Any serious incident that has occurred in relation to this device should be reported to BSN medical GmbH and the competent authority of your state.

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Manufactured for: BSN medical GmbH Quickbornstrasse 24 · 20253 Hamburg · Germany www.bsnmedical.com