Cutimed® Gel

D

Produktbeschreibung

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ENG

Product description

Cutimed® Gel is a transparent, amorphous hydrogel. It supports a moist wound environment and thereby contributes to autolytic wound cleansing.

Ingredients

Carbomer 940, purified water, glycerol, sodium hydroxide, EDTA sodium.

Indication

Cutimed® Gel supports autolytic cleansing of necrotic or unclean wounds (e.g. lightly weeping ulcer cruris, diabetic foot syndrome or decubital ulcers). Additionally, it can be used to produce a moist, physiological wound environment in later wound healing phases.

Warning

Cutimed® Gel is packaged for single use. Do not re-use or re-sterilise as there is a risk of transmission of body fluids or contaminated tissue between patients. Cutimed® Gel is intended for external application only. Cutimed® Gel may also be used on an infected wound in addition to an antimicrobial therapy under medical supervision.

Effect

By releasing moisture, Cutimed® Gel macerates necroses and wound coating and facilitates effective tissue-conserving debridement of the wound.

As a result, the wound may appear to become larger during initial changes of dressing, however, this is a sign that the healing process has begun.

Contraindications

Known hypersensitivity to the gel or individual ingredients

Notes on application

Cutimed® Gel may be applied either

- A • straight from the tube
- B • using a spatula
- C • using a sterile applicator tip, e.g., on a deeper wound

(1) Open the cap and (2) remove the sterile seal.

If required, (3) remove the applicator tip from the sterile packaging and (4) screw it onto the nozzle of the tube.

First irrigate the wound with Ringer's solution, physiological saline solution, etc. Then apply the Cutimed® Gel, completely covering the necrotic wound surfaces. In order to maintain the physiological, moist wound environment, cover the wound with a suitable secondary dressing.

Changing the dressing

To clean necrotic or unclean wounds we recommend that the dressing be changed and Cutimed® Gel reapplied after 3 days. In the case of clean wounds in the granulation or epithelial phase, the optimal time to change the dressing should be judged according to the exudate quantity.

At the latest, the dressing should be changed when the secondary dressing is no longer waterproof or has become saturated.

Notes on storage

Store at room temperature and protect from frost.

The hydrogel and applicator are gamma-sterilized.

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