
Cutimed® Hydro L & Cutimed® Hydro B

Bei beschädigter Verpackung nicht verwenden. Do not use if package is damaged. No pas utiliser si l'emballage est endommagé. No usar si el envase presenta daños. Non utilizzare se la confezione è danneggiata. Bij beschadigde verpakking niet gebruiken.

STERILE R

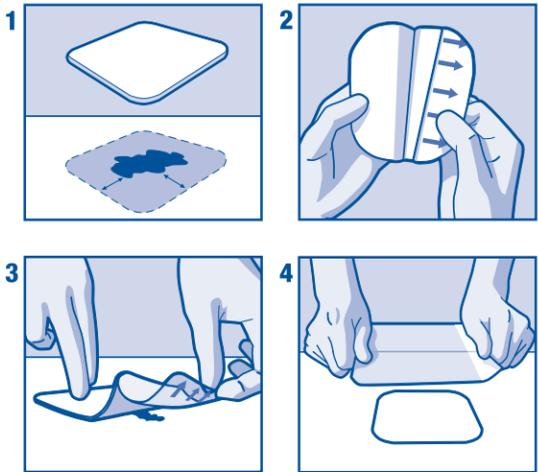Made and sterilized in USA

BSNmedical

® = registered trademark

BSN medical GmbH
Quickbornstrasse 24
20253 Hamburg · Germany

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Cutimed® Hydro L e Cutimed® Hydro B

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(ENG)

Product Description

Cutimed® Hydro is a range of sterile hydrocolloid wound dressings that help to maintain a moist wound environment. The dressing material interacts with the wound exudate to form a soft gel. Due to the matrix formulation of the hydrocolloid material, most of the gel is removed together with the dressing, resulting in little or no damage to the newly formed tissue. Cutimed® Hydro dressings are waterproof and remain in place during showering. They protect the wound against external contamination. Cutimed® Hydro dressings consist of a wound contact layer of hydrocolloids. The top layer is either a semi-permeable polyurethane film (Cutimed® Hydro L), or a film-coated polyurethane foam (Cutimed® Hydro B). Cutimed® Hydro B has a top layer of foam, and has tapered edges all around the product. Cutimed® Hydro L is conformable and has a top layer of low friction film, which allows close and easy monitoring of the healing process without dressing removal. Cutimed® Hydro dressings absorb exudate whilst maintaining a moist wound environment. By releasing moisture, Cutimed® Hydro facilitates autolytic debridement. As a result, the wound may appear to become larger during initial changes of the dressing.

Indications

Cutimed® Hydro B is primarily indicated for the management of low to moderately exuding chronic wounds. Under the supervision of a healthcare professional, Cutimed® Hydro B dressings may be used on:

- leg ulcers (ulcers of mixed aetiology)
- diabetic ulcers
- venous ulcers
- arterial ulcers
- pressure ulcers
- post-operative surgical wounds
- skin donor sites

Cutimed® Hydro L is primarily indicated for management of superficial, dry to low exuding wounds such as:

- lacerations
- minor abrasions
- minor burns (superficial and partial thickness)
- post op protection of closed surgical sites

Cutimed® Hydro dressings are suitable for use under compression bandages.

Contraindications

- Cutimed® Hydro dressings should not be used:
- on patients with a known sensitivity/allergy to any of the product components.
 - on third-degree burns.

Warnings and precautions

Cutimed® Hydro is packaged for single use. Do not re-use or re-sterilise as this is a risk of transmission of body fluids or contaminated tissue between patients. Do not use if pouch is open or damaged. Discard open or unused material of the device. The wound should be inspected for signs of infection and -in case- treated according to clinical practice. In case of any signs of adverse reactions, macerations (whitening of skin), hypergranulation (excess issue formation), irritation (reddening, inflammation), infection (increased pain, bleeding, wound drainage), change in odor or color of wound exudate, sensitivity (allergic reaction) of the wound or prolonged enlargement of the wound do not continue the application and consult a healthcare professional.

Contact also a healthcare professional if:

- The wound does not begin to show signs of healing or any other unexpected symptoms occur. This wound dressing can be used with other wound care products when consulting a healthcare professional. Dispose of the used dressing as per local regulation.

For external use only.

Directions for use

Wound preparation

- Prepare the wound according to wound management protocol.
- Remove excess moisture from surrounding skin.

Dressing Application

- Choose a dressing size and shape to ensure that the dressing is 2.5 cm (1") larger than the wound area in any one direction.
- Remove the dressing from the packaging and partially remove the white release paper exposing the center of the dressing. Then grasp the clear release film at the end with the arrows.
- Position the dressing over the wound and line up the center of the dressing over the center of the wound. Gently press the dressing in place over the wound.
- Remove the white release paper completely and secure the adhesive to the skin.
- Next remove the clear release film from the other side of the dressing and secure the adhesive to the skin.
- Place your hand over the dressing and mold the dressing into place for at least one minute.

Difficult to Dress Wounds

Difficult to dress wounds like sacral areas need special attention.

- Remove the dressing from the package and remove the white release paper from the back of the dressing.
- Grasp the clear release film with the arrows at each edge of the dressing.
- Hold the adhesive side of the dressing over the wound and line up the center of the dressing over the center of the wound. Position the "hinge" of the dressing into the area between the buttocks and gently press the dressing in place over the wound.
- Remove the clear release film from each end of the dressing and secure to the skin.
- Place your hand over the dressing and mold the dressing into place.

Note: The dressing should be inspected frequently for leakage and bunching/rolling up of the edges. If any of these occur, the dressing should be changed. As wound fluid is absorbed by the dressing, gel formation may be visible from the outer surface of the dressing.

Dressing Change and Removal

- Cutimed® Hydro can remain in situ up to 7 days, depending on patient condition and the level of exudate. Frequent dressing changes on wounds with damaged or delicate skin surrounding the wound is not recommended.
- Press down on the skin with one hand and carefully lift an edge of the dressing with your other hand.
- Repeat this procedure around the perimeter of the dressing until all the edges are free and gently roll the dressing off of the wound site.
- If required, irrigate the wound site with a suitable wound cleanser prior to application of new dressing. It is not necessary to remove all dressing residue -if any present-, since this is not harmful to the healing process.

Infected Wounds

Cutimed® Hydro may be used on an infected wound in addition to antimicrobial therapy under medical supervision.

Storage

Cutimed® Hydro should be stored in dry conditions below 25°C/77°F. Do not expose to direct sunlight. Avoid refrigeration and exposure to high humidity.

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