Instructions for Use

ENG

**Cutimed® Siltec® PLUS Heel****Product description**

Cutimed® Siltec® PLUS Heel dressings are an assortment of silicone coated foam dressings. Cutimed® Siltec® PLUS Heel is a sterile, absorbent polyurethane foam dressing for traumatic dressing changes that contains super-absorbent stripes which absorb and lock wound exudate. The wound contact surface is a perforated silicone layer that allows adherence to the peri-wound skin but not to the moist wound bed or to newly formed epithelial tissue. This minimises trauma and pain during dressing changes. The outer film is water-repellent, yet permeable to oxygen and vapour. The foam dressing and the super-absorbent stripes are designed to absorb and to lock away excess wound fluid inside the dressing, promoting a moist wound environment and minimising the risk of maceration. Cutimed® Siltec® PLUS Heel is highly conformable and provides protection of the wound site. Cutimed® Siltec® PLUS Heel leaves no residues in the wound. The dressing reliably retains wound exudate, even under compression. If necessary the dressing can be cut to size. Cutimed® Siltec® PLUS Heel is anatomically shaped for the treatment of heel wounds, gently adheres to the peri-wound skin and stays in place while secondary fixation is applied.

**Indications**

The Cutimed® Siltec® PLUS Heel assortment is indicated for exuding wounds such as: venous and arterial ulcers, pressure ulcers, diabetic gangrenes, surgical incisions, skin grafts and donor sites, lacerations or abrasions. Cutimed® Siltec® PLUS Heel is recommended for wounds with low to high exudate levels.

**Contraindications**

Do not use on patients with a known sensitivity to any of the product components.

**Warnings and precautions**

Cutimed® Siltec® PLUS Heel is packaged for single use. Do not re-use or re-sterilise as there is a risk of transmission of body fluids or contaminated tissue between patients. Do not use if pouch is open or damaged. Discard open or unused material. The wound should be inspected for signs of infection and treated according to clinical practice if required. For external use only. Do not use with oxidising solutions such as hypochlorite or hydrogen peroxide. Remove the dressing prior radiation therapy.

**Directions for use**

- Clean the wound according to good clinical practice.
- Make sure, the surrounding skin is dry.
- Choose an appropriate dressing size that covers the wound and overlaps the wound edges. **(A)**
- Open the pouch and take out the dressing. Ensure sterility.
- Remove protective film **(B)** and apply with the adherent side towards the wound. **(C)**
- Avoid stretching the dressing.
- Apply additional dressing fixation.
- If necessary, Cutimed® Siltec® PLUS Heel can be used in combination with other dressings e.g. Cutimed® Sorbact® (in case of infected wounds) or Cutimed® Gel (for dry, necrotic wounds).
- Cutimed® Siltec® PLUS Heel may be used on an infected wound in addition to antimicrobial therapy under medical supervision.
- Cutimed® Siltec® PLUS Heel can remain in place for up to seven days, depending on wound conditions and exudate levels. It is recommended to change the dressing every 24 hours initially, moving to less frequent changes as necessary.
- Carefully remove any additional dressing fixation and gently remove Cutimed® Siltec® PLUS Heel from the wound and discard.
- If required, cleanse the wound prior to application of a new dressing.

**Storage**

Cutimed® Siltec® PLUS Heel should be stored in dry conditions. Do not expose to direct sunlight.

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**BSN medical GmbH**