ENG Bacteria binding Round Swabs

Device Description

Cutimed® Sorbact® Round Swab is a sterile, bacteria and fungi binding wound dressing. It consists of a green Sorbact® wound contact layer shaped to a spherical swab, held together by a silicone ring.

Mode of Action

Cutimed® Sorbact® binds microorganisms, such as Staphylococcus aureus (including MRSA), Streptococcus species, Escherichia coli, Pseudomonas aeruginosa and Candida albicans, as shown in vitro. These microorganisms are removed from the wound each time the dressing is changed.

Intended Use

Cutimed® Sorbact® Round Swab is suitable for wound cleansing or as a wound filler in the management of clean, contaminated, colonized or infected exuding wounds, such as minor cavity wounds.

Contraindications

There are no known contraindications to the use of Cutimed® Sorbact® Round Swab. Attention is drawn to the warnings and precautions.

Warnings and Precautions

If the treated condition deteriorates, fails to improve or a side effect is observed, consult a physician or an appropriate health care provider.

Do not use on patients with a known hypersensitivity to any of the dressing components.

The dressing should always be applied in direct contact with the wound area to allow microorganisms to bind to the dressing.

Should the dressing adhere to the wound, moisten or soak the dressing to assist removal and avoid disruption of the healing wound.

Do not use in combination with fatty products, such as ointments, creams and solutions as they may decrease the binding of microorganisms.

Cutimed® Sorbact® Round Swab is for single use only and should not be re-used. Re-use may lead to increased risk of infection or cross-contamination.

Do not re-sterilize.

Do not cut the dressing.

Do not use if the peel pouch is already open or damaged as the sterility of the dressing is guaranteed only when the package is unopened and undamaged prior to use.

Prior to radiation therapy remove Cutimed® Sorbact® Round Swab. A new dressing can be applied following treatment.

Cutimed® Sorbact® Round Swab does not contain x-ray thread and should not be used in open surgery.

Cutimed® Sorbact® Round Swab is magnetic resonance imaging (MRI) safe.

Instructions for Use (wound cleansing)

1. Remove the dressing from the pouch using an aseptic technique.

2. Moisten Cutimed® Sorbact® Round Swab with an irrigation solution (e.g. saline solution); remove any excess irrigation solution.

3. Gently clean the wound according to local clinical practice.

Instructions for use for wound management

1. Prepare the wound according to local clinical practice.

2. Remove the dressing from the pouch using an aseptic technique.

3. Discard any open and unused dressing.

4. In cavity wounds, apply the dressing to ensure direct contact with the complete wound surface.

5. Direct wound contact is a prerequisite for the effective binding of microorganisms. Moist wound conditions are essential to ensure the efficacy of Cutimed® Sorbact® and to avoid potential sticking to the wound.

6. Depending on the amount of wound exudate, Cutimed® Sorbact® Round Swab can be covered with all types of hydroactive secondary dressings such as foams, superabsorbent dressings or alginate.

7. The dressing change frequency depends on exudate levels and the overall condition of the wound and surrounding skin. Should the clinical condition allow, the dressing can be left in place for up to 7 days. The final decision on the therapy regimen needs to be taken by health care professionals.

Storage and Disposal

Keep dry and away from sunlight. Disposal should be made according to local environmental procedures.

Notice

Any serious incident that has occurred in relation to this device should be reported to BSN medical GmbH and the competent authority of your state.

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