ENG Bacteria binding ribbon gauze

Device Description

Cutimed® Sorbact® Ribbon Gauze is a sterile, bacteria and fungi binding wound dressing. It consists of a green Sorbact® wound contact layer, which allows passage of wound exudate into a secondary dressing. The dressing can be used in compression therapy.

Mode of Action

Cutimed® Sorbact® binds microorganisms, such as *Staphylococcus aureus* (including MRSA), *Streptococcus* species, *Escherichia coli*, *Pseudomonas aeruginosa* and *Candida albicans*, as shown in vitro. These microorganisms are removed from the wound each time the dressing is changed.

Intended Use
Cutimed® Sorbact® Ribbon Gauze is intended for use in the management of clean, contaminated, colonized or infected exuding wounds, such as cavity wounds and fistulas. Cutimed® Sorbact® Ribbon Gauze is also intended for use in fungal infections in skin folds.

Contraindications

There are no known contraindications to the use of Cutimed® Sorbact® Ribbon Gauze. Attention is drawn to the warnings and precautions.

Warnings and Precautions

If the treated condition deteriorates, fails to improve or a side effect is observed, consult a physician or an appropriate health care provider.

Do not use on patients with a known hypersensitivity to any of the dressing components. The dressing should always be applied in direct contact with the wound area to allow microorganisms to bind to the dressing.

Should the dressing adhere to the wound, moisten or soak the dressing to assist removal and avoid disruption of the healing wound.

Do not use in combination with oily products, such as ointments, creams and solutions as they may decrease the binding of microorganisms.

Cutimed® Sorbact® Ribbon Gauze is for single use only and should not be re-used. Re-use may lead to increased risk of infection or cross-contamination.

Do not re-sterilize.

Do not use if the pouch is already open or damaged, as the sterility of the dressing is guaranteed only when the package is unopened and undamaged prior to use.

Prior to radiation therapy, remove Cutimed® Sorbact® Ribbon Gauze. A new dressing can be applied following treatment.

Cutimed® Sorbact® Ribbon Gauze does not contain x-ray thread and should not be used in open surgery.

Cutimed® Sorbact® Ribbon Gauze is magnetic resonance imaging (MRI) safe.

Instructions for use for wound management

1. Prepare the wound according to local clinical practice. If desired, gently press a Cutimed® Sorbact® round swab against the moist wound surface to remove superficial microbial contamination.

2. Select an appropriate dressing size for the wound.

3. Remove the dressing from the pouch using an aseptic technique.

4. If cutting the dressing, use an aseptic technique. Discard any open and unused dressing.

5. Direct wound contact is a prerequisite for the effective binding of microorganisms. Always apply the dressings in a way that ensures direct contact with the wound surface. Moist wound conditions are essential to ensure the efficacy of Cutimed® Sorbact® and to avoid potential sticking to the wound.

6. In cavity wounds and fistulas, unfold and fluff up one or more dressings and line/fill the cavity or fistula. Avoid dense packing. Always leave a part of the dressing protruding so that it can be easily removed. If cutting the dressing, it is recommended to leave the cut end outside the cavity or fistula to avoid residual threads in the wound.

7. Depending on the amount of wound exudate, Cutimed® Sorbact® ribbon gauze can be covered with all types of hydroactive secondary dressings such as foams, superabsorbent dressings or alginate.

8. The dressing change frequency depends on exudate levels and the overall condition of the wound and surrounding skin. Should the clinical condition allow, the dressing can be left in place for up to 7 days. The final decision on the therapy regimen needs to be taken by health care professionals.

Instructions for use for fungal infections

1. Prepare the affected area according to local clinical practice.

2. Do not use antifungal creams or ointments in combination with the dressing.

3. Select an appropriate dressing size to be able to cover the affected area.

4. Apply the dressing, cut if necessary. Ensure that the dressing comes into direct contact with the affected area.

5. Dressings can be secured with undergarments if necessary. The use of a skin friendly adhesive is recommended if additional fixation is needed. The adhesive should be applied outside the affected area.

6. Change the dressing once daily in conjunction with inspection.

7. Continue to use the dressing for 2-3 days after the symptoms have resolved to avoid recurrence.

Storage and Disposal

Keep dry and away from sunlight. Disposal should be made according to local environmental procedures.

Notice

Any serious incident that has occurred in relation to this device should be reported to BSN medical GmbH and the competent authority of your state.

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